

## SULCI PROTECTORS

Sulci protectors are ideal for straightening, mending and shaping the side edges of the nail and for cushioning the nail against irritation or inflammation from the nail wall. They offer a distinct advantage over packing, especially for ingrown toenails.

They can be left in place for up to eight weeks (must be re-fixed after four weeks). The splints do not just provide a cushion, they also have the effect of maintaining the normal width and shape of the nail at the free edge. That prevents changes to the nail bed, protrusion of the nail wall and the

resulting deformities and growth disorders of the nail.

Sulci protectors are available in three different sizes. Sulci protectors can be positioned using tweezers or a Sulci injector. GEHWOL PROFESSIONAL Nail Repair Gel is the ideal fixative for the protector.



1. With ingrown toenails, the side edge of the nail often has sharp edges or spikes that press deep into the fold, causing a painful inflammatory reaction.



2. The edge of the nail is first straightened and smoothed with the help of e.g. a file.



3. The small splints of soft plastic open lengthwise. They are cut to the correct length and the short edge is smoothed.



4. After the nail has been disinfected and any grease has been removed, the Sulci protector is pushed over the free side edge of the nail using the tweezers.



5. The finished Sulci protector.



6. The visible part of the Sulci lying on the nail plate is fixed in place with GEHWOL PROFESSIONAL Nail Repair Gel.

## SULCI-INJECTOR MK 2

Product details:

- For inserting protectors into the Sulci



Medical Device

Order no.

313 370 001

MD

•